


Inserting the probe into the CMS-50F



Since the probe connection is round, there is 1 pin that appears to be broken off inside of the connection. This is normal and comes from the factory like this. This keeps you from installing the connector in the wrong position.



It is hard to see in this picture, but there is 1 pin that is missing in the wrist oximeter.

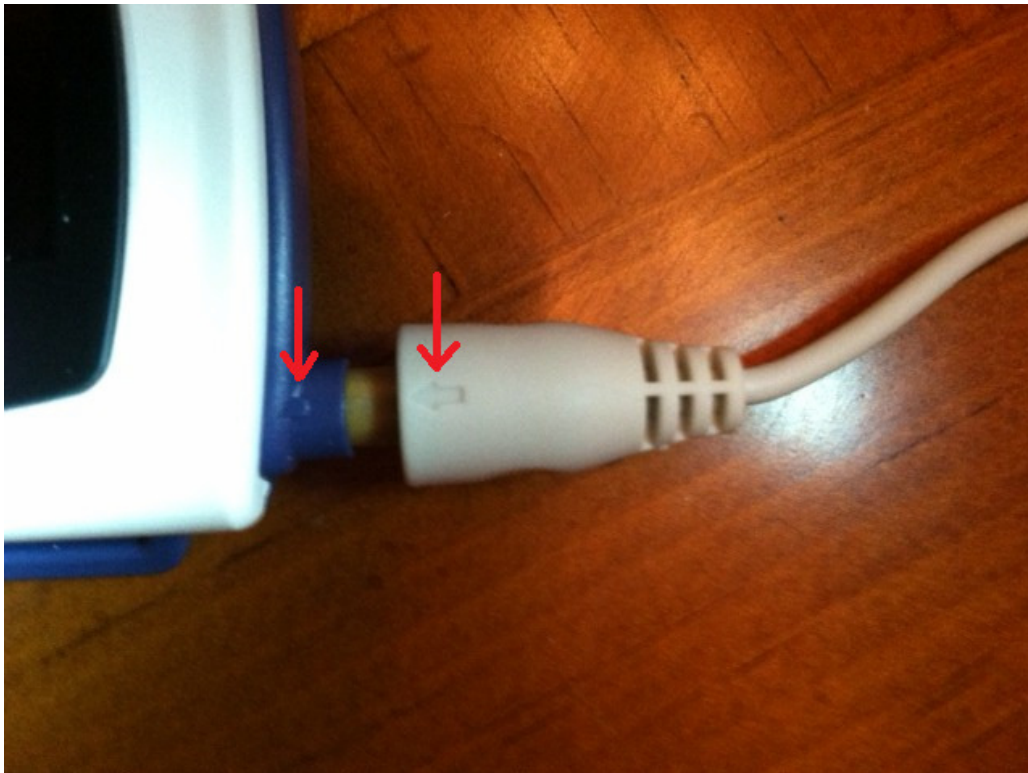

All you have to do is line the arrow on the probe with the little line on the wrist unit and gently push. If you feel resistance, stop and line up the probe and unit again.



The probe should slide in with very little resistance. Do not push or force the probe in place. This will result in bent pins inside the wrist unit. These can be bent back in place with a small screwdriver if they become bent.

